

## Sit-to-Stand Assist

First apply central locking wheel brakes

Footplate can be in either in up or down position

Handset controller will raise and lower seat

This feature can be used over the toilet, bed -side or in the shower



## I.V. Pole Receptacles / Storage Area



# Stretchair<sup>®</sup>

*Safe Versatile Patient Care*



1000 lb. Patient Weight Capacity

Bedside or Over-the-Toilet Commode  
Powered Sit-to-Stand Seat Assist - Shower Chair  
Transport Chair

# Stretchair<sup>®</sup>

*Safe Versatile Patient Care*

8110 Ulmerton Road  
Largo, Florida USA 33771-3944  
800-237-1162 727-536-0666 FAX  
[www.stretchair.com](http://www.stretchair.com)

© 2006 Stretchair Patient Transfer Systems, Inc.  
All rights reserved.



Proudly  
Made in the USA

## Shower / Bath Use

Transport patient to shower with foot -plate down  
Engage steer lock for long transports

**In the shower stall always apply central wheel brakes**  
**Hand-rails should be use for patient safety**

Power sit-to-stand feature may be used in shower if desired

Pat dry Hygiene-Chair before transporting back to room

Do not plug-in external charging unit in shower area



## Over-the-Toilet Use

**Note: If battery is dead receptacle can be removed/inserted by manually lifting the seat from behind.**

- 
- tacle by raising seat w/ handset
- Storage hook for receptacle is located on side
- Wheel Hygiene-Chair over top of standard toilet

Remove recep-



## Bed-side Use

- Must raise seat with handset control to insert receptacle pan
- **Must apply central locking wheel brakes for safety**



## Battery / Control Box



Display shows battery strength  
(25%, 50%, 75%, 100%)

Red Button must be in "out" position for operation

Press Red Button in for emergency shut -down

Battery strength should be kept at 50% or better to ensure consistent, worry free operation

**Must use external charger for charging battery**

## External Battery Charging Unit

- Plug external charger into 110volt outlet
- Slide battery until locked into place on charger
- Charge light and on light will be lit during charging
- When battery is fully charged the charge light will go out
- Press battery release handle to remove battery
- Use external charger in safe, dry area only

